

USER INSTRUCTIONS

FASTRS

- Position the wheelchair facing forwards centrally over the rails and apply the brakes
- Attach the front tie-down straps to the main frame of the wheelchair, ensuring the rail clips are locked in the rails with the straps at an angle of approximately 45 degrees
- Fit the rear tie-down straps into the rails behind the wheelchair, ensuring the rail clips are locked in the rails
- Pull the webbing from the rear tie-down retractors by pressing the red release levers and attach the straps to the main frame of the wheelchair, so that an angle of approximately 45 degrees is achieved. Tension with the handles
- Clip the TITUS passenger harness into the rails behind the rear tie-down straps (with the stalk to the inside of the vehicle), ensuring the rail clips are locked in the rails
- Fit the lap belt (black webbing) around the occupant's pelvis and clip into the stalk
- Fit the shoulder belt (blue webbing) over and across the occupant's shoulder and clip into the tongue on the lap belt
- Adjust either the Comfort Clip or 3-point belt (depending on which is fitted/being used) for occupant comfort
- To remove, reverse above procedure



Tel: 01935 426695 • Fax: 01935 433766
Email: enquiries@koller.co.uk • Web: www.koller.co.uk