

USER INSTRUCTIONS

EXCEL120 combined

- Position the wheelchair centrally over rails and apply the brakes
- Attach the front tie-down straps to the wheelchair, ensuring the clips are locked in the rails
- Attach the rear tie-down straps (with the stalk to the inside of the vehicle) to the wheelchair, ensuring the clips are locked in the rails
- Tension the system by pulling the webbing through the over-centre buckles while closing them
- Fit the lap belt (black webbing) around the occupant's pelvis, and clip into the stalk
- Fit the shoulder belt (blue webbing) over and across the occupant's shoulder, and clip into the tongue on the lap belt
- Adjust either the Comfort Clip or 3-point belt (depending on TITUS harness fitted) for occupant comfort
- To remove, reverse above procedure



Tel: 01935 426695 • Fax: 01935 433766
Email: enquiries@koller.co.uk • Web: www.koller.co.uk

Copyright © 2015 Koller Engineering Ltd. All rights reserved. Issue 3 July 2015

