


USER INSTRUCTIONS

GENESIS⁸⁵

- Position the wheelchair centrally over the rails and apply the wheelchair brakes
- Attach the front tie-down straps to the wheelchair, ensuring the clips are locked in the rails
- Attach the rear tie-down straps to the wheelchair, ensuring the clips are locked in the rails, then tension by pulling the webbing through the over-centre buckles while closing them
- Clip the lap belt (black webbing) into a front Mini Buckle on one rear strap, place around the occupant's pelvis, clip into the corresponding Mini Buckle on the other rear strap and pull the webbing through the adjustable tongue until "snug"
- Fit the shoulder belt (blue webbing) over and across the occupant's shoulder, clip into the free Mini Buckle on the other rear strap and pull the webbing through the adjustable tongue until "snug"
- If a 3-point belt (upper anchorage) is used, fit to the free Tongue on the Shoulder Belt and adjust for comfort
- To remove, reverse above procedure

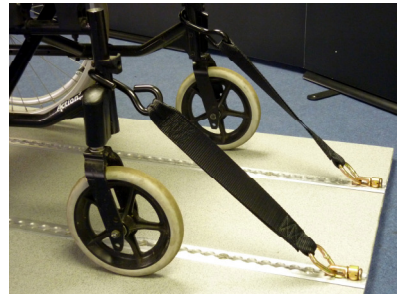

Tel: 01935 426695 • Fax: 01935 433766
Email: enquiries@koller.co.uk • Web: www.koller.co.uk

Copyright © 2015 Koller Engineering Ltd. All rights reserved. Issue 1 July 2015



KOLLER