

USER INSTRUCTIONS

- Position the wheelchair centrally over rails and apply brakes
- Attach the front tie-down straps, ensuring the clips are locked in the rails
- Attach the rear tie-down straps (with the stalk to the inside of the vehicle), ensuring the clips are locked in the rails
- Tension the system by pulling the webbing through the over-centre buckles while closing them
- Fit the lap belt (black webbing) around the occupant's pelvis, ensuring it is locked in the stalk
- Fit the shoulder belt (blue webbing) over and across the occupant's shoulder, ensuring it is locked into the tongue on the lap belt
- Adjust either the Comfort Clip or 3-point belt (depending on TITUS harness fitted) for occupant comfort
- To remove, reverse above procedure

For detailed user instructions, refer to User Instruction manual, or visit:

www.koller.co.uk

