

USER INSTRUCTIONS

FAST

- Position the wheelchair centrally over the rails and apply the wheelchair brakes
- Attach the front tie-down straps to the wheelchair main frame (press the release levers to pull the webbing from the retractors) and fit into the rails, ensuring the clips are locked in the rails with straps at an angle of approximately 45 degrees
- Fit the rear tie-down straps into the rails behind the wheelchair, ensuring the clips are locked in the rails
- Pull the webbing from the rear tie-down retractors (press the release levers) and attach to the wheelchair main frame so that an angle of approximately 45 degrees is achieved. Tension with the handles
- Clip the TITUS passenger harness into the rails behind the rear tie-down straps (with the stalk to the inside of the vehicle), ensuring the clips are locked in the rails
- Fit the lap belt (black webbing) around the occupant's pelvis and clip into the stalk
- Fit the shoulder belt (blue webbing) over and across the occupant's shoulder and clip into the tongue on the lap belt
- Adjust either the Comfort Clip or 3-point belt (depending on which is fitted/being used) for occupant comfort
- To remove, reverse above procedure



Tel: 01935 426695 • Fax: 01935 433766
Email: enquiries@koller.co.uk • Web: www.koller.co.uk

Copyright © 2016 Koller Engineering Ltd. All rights reserved. Issue 1 June 2016

