


# USER INSTRUCTIONS

## EXCEL APOLLO

- Position the wheelchair centrally in relation to the floor spigots
- Slide the “cut away” end of the housings of the front tie-down straps over the spigots (pushing down at the same time) and continue until the levers lock in place
- Attach the front tie-down straps to the main frame at an angle of approximately 45 degrees. Apply the wheelchair brakes
- Attach the rear tie-down strap housings to the spigots as above, ensuring the stalk is placed to the inside of the vehicle
- Release the over-centre buckles and pull the webbing through the rear tie-down straps. Attach to the wheelchair main frame so that an angle of approximately 45 degrees is achieved. Tension the system - pull the webbing through the over-centre buckles while closing them
- Fit the lap belt (black webbing) around the occupant’s pelvis and clip into the stalk
- Fit the shoulder belt (blue webbing) over and across the occupant’s shoulder and clip into the tongue on the lap belt
- Adjust either the Comfort Clip or 3-point belt (depending on which is fitted/being used) for occupant comfort
- To remove, reverse above procedure

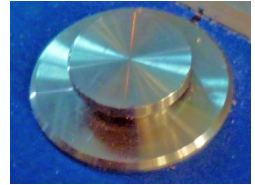

Tel: 01935 426695 ● Fax: 01935 433766  
Email: [enquiries@koller.co.uk](mailto:enquiries@koller.co.uk) ● Web: [www.koller.co.uk](http://www.koller.co.uk)

Copyright © 2016 Koller Engineering Ltd. All rights reserved. Issue 1 July 2016

