

USER INSTRUCTIONS

FALL STOP

These instructions cover the use of the FALL STOP tail lift barrier. No tools are required to fit or remove FALL STOP.



Red Locking Handle
in "locked" position



Locking Rail



Fitting

When the tail lift is on the ground to accept the wheelchair passenger, ensure the wheelchair passenger is positioned on the tail lift with the brakes **locked** before fitting FALL STOP. If the tail lift is at vehicle floor height to accept the wheelchair passenger, fit FALL STOP **first** then manoeuvre the wheelchair passenger onto the tail lift, ensuring the brakes are **locked**, before operating the tail lift.

- With the red locking handle in the **locked** position, insert FALL STOP into the top of the locking rail and slide towards the middle, until it **locks**.
- Attach one end of one strap to FALL STOP by inserting the hook through one of the frame holes. Clip the hook at the opposite end of the strap around the tail lift grip rails. Remove slack in the strap by pulling the loose end of the strap through the black adjuster. Repeat on the other side

Removing

Only remove FALL STOP when the wheelchair passenger is safely secured in the vehicle or on the ground.

- Slacken one strap by feeding the strap through the adjuster, just enough to remove the hooks from FALL STOP and the tail lift grip rail. Repeat on the other side
- Twist the red locking handle anti-clockwise until it stops (it will lift and turn 90 degrees)
- Slide FALL STOP 1/2" in either direction until it lifts clear of the locking rail
- Store FALL STOP and straps securely in the vehicle