
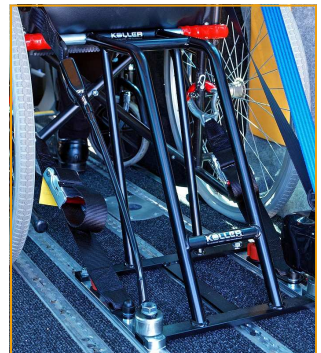


USER INSTRUCTIONS

- Position the wheelchair centrally over rails and apply brakes
- Locate IRiS in both rails as close to the back of the wheelchair as possible and slide slightly forward, ensuring it locks in **BOTH** rails
- Pull out the handles so that they extend past the upright tubes of the wheelchair
- Open the over-centre buckles and extend the webbing to attach the karabiners to the wheelchair. Tension the system by pulling the webbing through the over-centre buckles while closing them
- Fit the lap belt (black webbing) around the occupant's pelvis, ensuring it is locked in the stalk
- Fit the shoulder belt (blue webbing) over and across the occupant's shoulder, ensuring it is locked into the tongue on the lap belt
- Adjust either the Comfort Clip or 3-point belt (depending on TITUS harness fitted) for occupant comfort
- To remove, reverse above procedure

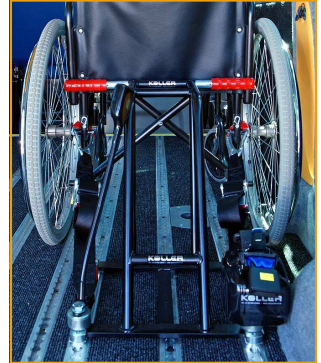

For detailed user instructions, refer to User Instruction manual, or visit:

www.koller.co.uk

